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### (EN) Directions for use - Bowie-Dick single use test pack

The stericlin® Bowie-Dick single use test pack serves to carry out the performance test in steam sterilisers. The stericlin® Bowie-Dick single use test pack consists of pigments not containing any metals. The indicator system is odorless and does not lose colour. To provide perfect performance of the highly sensitive indicator, these directions for use should be read and followed carefully. The indicator system meets the requirements of ISO 11140-4, the indicator sheet is a type 2 indicator according to ISO 11140-1, the indicator dot outside is a type 1 indicator according to ISO 11140-1.

1. Before starting the test run, warm up the steriliser with an empty chamber run.
2. Put the stericlin® Bowie-Dick single use test pack horizontally on a low area of the chamber of the steriliser, e. g. above the drain of the empty chamber or nearby the door of the chamber.
3. Start the Bowie-Dick test run. After finishing the complete test run (134°C. for 3,5 minutes \*) the single use test pack should be removed immediately from the chamber. Open the test pack, take out the test sheet and examine and document the result.
4. Examine the test sheet under top light. **Evaluation of the test sheet with bottom light or a light source from the background is not appropriate.** Compare the indicator's colour shade at the edge of the test sheet with the indicator colour in the middle of the test sheet.

If the steriliser manages air removal and steam penetration effectively and if steam quality is within specification (maximum concentration of inert gases not exceeded), the indicator colour changes completely and uniformly from blue to pink.

If steam penetration or steam quality (high concentration of inert gases) are insufficient, the colour change will be incomplete and not uniform. In that case the colour in the middle of the test sheet is different (blue or violet) to the colour at the outer areas of the test sheet. **In this case check your steriliser.**

5. Documentation of the result with signatures of the operator and supervisor are sufficient under legal aspects. Archiving the test sheet itself is not necessary.

\*) Temperature and time can slightly vary according common tolerances; please see e.g. EN 285 and EN 13060.

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