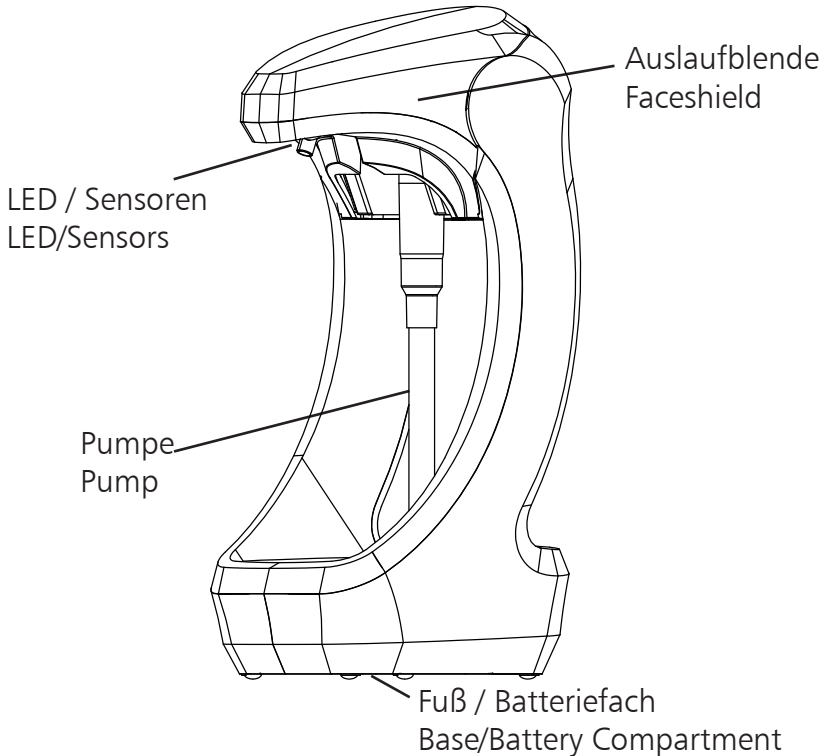
RX5 - T

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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1.	Indications for the use of the service instructions ..	16
2.	Intended use.....	16
3.	Safety information.....	17
4.	Technical drawing.....	18
5.	Scope of delivery.....	18
6.	Mounting of the dispenser.....	19
7.	Going into operation.....	21
8.	Basic operation.....	23
9.	Changing stroke.....	24
10.	LED Function.....	24
11.	Maintenance / attendance and maintenance.....	25
12.	Eliminate trouble.....	26
13.	Warranty.....	27
14.	Disposal.....	27
15.	Technical data.....	28

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1. Indications for the use of the service instructions

The following symbols are used in the instructions

Safety reference:

A non-observance may cause personal injury

NOTICE

Indicates a property damage message

2. Intended use

This appliance is suitable for hand disinfectant and soaps.

It is **not** intended for use with high-viscosity products or abrasive and aggressive agents.

The dispenser is for free standing, wallmount or pillar mount use. Use this appliance indoors only.

Use the appliance for hand disinfection and cleaning, under normal use cycles.

Ambient conditions:

-  • Never use the appliance in potentially explosive areas.
-  • Only use the appliance in a dustless environment.
-  • Pay attention to that the apparatus is only used at a relative air moisture of <90% (not condensing).
-  • Only use this apparatus at room temperature 15 -30 °C

EN 3. Safety information

3. Safety information

Attention!

Please read through carefully these operating instructions prior to initial operation and keep it.

OPHARDT hygiene does not assume any liability for damages in the case of:

- defects at the appliance caused by mechanical influences
- changings at the appliance without express approval of **OPHARDT hygiene**
- use for other purposes than that mentioned in the application

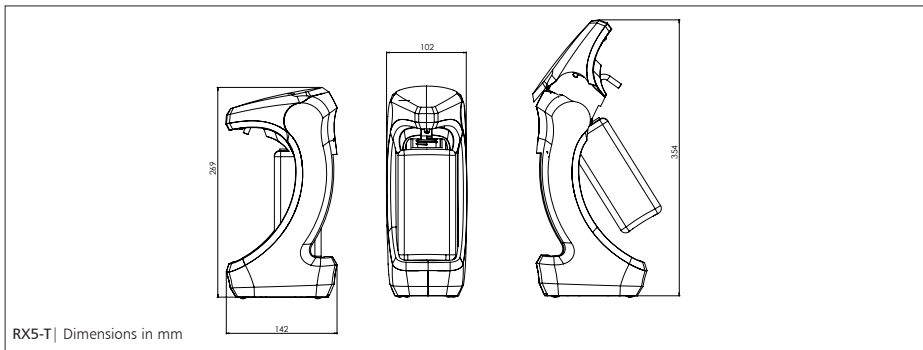
3.1 General safety

- ⚠ • Repairs at the appliance may exclusively be carried out by **OPHARDT hygiene** employees or persons authorized by **OPHARDT hygiene** Important dangers may occur by unexpert repairs. In case of a necessary repair please contact the manufacturer.
- ⚠ • Suspend the appliance on a dry and splash-proof place.
- ⚠ • This appliance is not a toy. Keep and use it out of the reach of children. The appliance should be installed where children cannot reach it.
- ⚠ • Persons with physical, sensory or mental impairment incl. children should be supervised when using this appliance.
- ⚠ • The appliance is not suitable for the transportation of media.
- ⚠ • Do not place the device directly in trafficked routes.
- ⚠ • The appliance is not suitable for the metering of other than the indicated media.
- ⚠ • Exclusively operate the appliance with leakproof alcalic batteries of the same manufacturer and type and with the same charging condition. Only use batteries with the indicated size type C.
- ⚠ • Do not operate the appliance with rechargeable batteries (accumulators).
- ⚠ • Do not use the dispenser if batteries have leaked in the appliance.
- ⚠ • Never use force.
- ⚠ • Take the batteries from the appliance prior to cleaning and attendance.
- ⚠ • The appliance need not be disassembled, no repairable parts.
- ⚠ • The face shield is not removable.
- ⚠ • The battery supply clamps may not be short-circuited with each other.
- ⚠ • Cleaning and user maintenance tasks must not be carried out by unsupervised children.

3.2 Safety during operation of the appliance

- ⚠️ • Protect your appliance against humidity.
- ⚠️ • Do not expose the appliance to thermal radiation and heat.
- ⚠️ • Never submerge the appliance in the water.
- ⚠️ • Only fill the appliance with the indicated substances in accordance with the intended use.
- ⚠️ • There are no dismantlable parts with the exception of pump, receptacle, battery cover, batteries
- ⚠️ • The appliance is not autoclavable and cannot be cleaned in the dishwasher.
- ⚠️ • The apparatus shouldn't be cleaned with water jets or by means of a high-pressure cleaner.
- ⚠️ • Please make sure that the medium does not reach the ground.

4. Technical drawing

5. Scope of delivery

- Dispenser
- Plastic pump
- Receptacle, 500 ml
- Batteries (4 mono-cells, type LR 14)
- Instruction manual
- Optional: Wallmount (including ingo-man® standard backplate and fixing set)

EN 6. Mounting of the dispenser

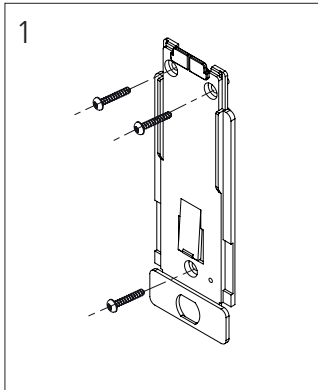
6. Mounting the dispenser

6.1 Wall mounting

The dispenser has the option of being free standing or wall mounted.

If free standing please ensure that:


- The surface is level and secure
- It is clear of any passing or moving objects
- If the dispenser is knocked any spillage should not effect near by equipment

If the dispenser is to be mounted on a wall please follow the instructions listed below:

NOTICE

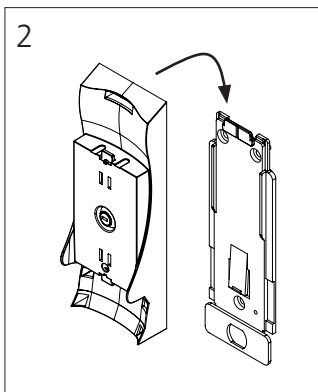
- The dispenser must be mounted to the fixing apertures provided on a non-sloping wall using at least 3 screws.
- Mount the appliance on the wall using the enclosed fixings or other suitable fixings.
- Check the suitability of the wall before mounting!
- When choosing where to mount the appliance, please consider ease of access for filling!
- Do not install the appliance in narrow entrances and passage-ways!
- Microwaves or fluorescent tubes can cause electromagnetic interference. If this is the case, please remove the source of the interference. This will restore the device to problem-free operation.


Fix the ingo-man backplate to the wall using the rawl plugs & screws provided (fig. 1).

Push the wall mount to the wall and slide down over ingo-man backplate until the spring tongue is engaged (fig. 2).

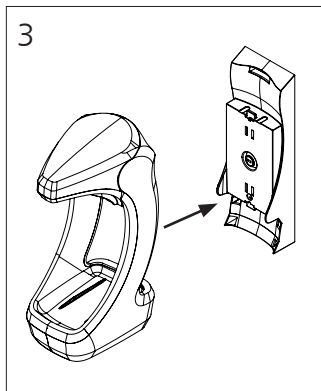
EN 6. Mounting of the dispenser

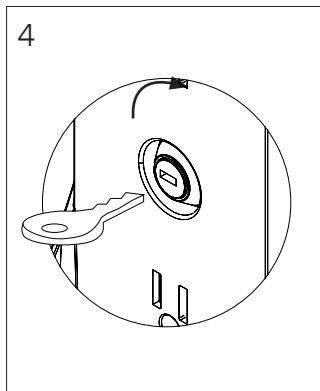Place the dispenser on the wallmount, lining up the bottle opening with the rectangular boss on the wallmount (fig. 3).

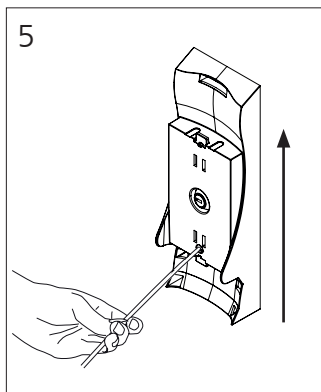
Turn the key to lock the dispenser in place. Remove the key before inserting the bottle (fig. 4).

To remove the wallmount from the ingo-man backplate, using a thin tool push on the tongue of the backplate (through the hole provided) and slide the wallmount upwards (fig. 5).

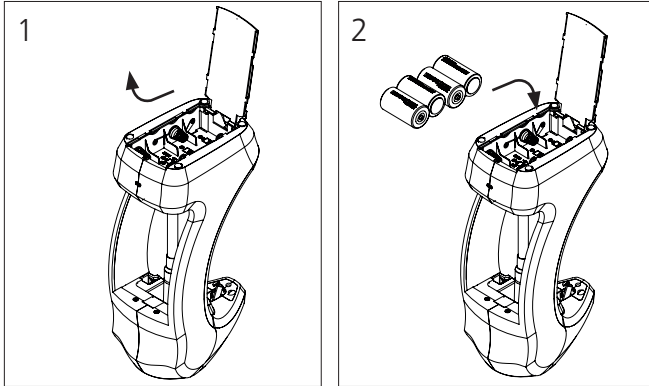EN 7. Going into operation

7. Going into operation

7.1 Insert/Replacing of the batteries

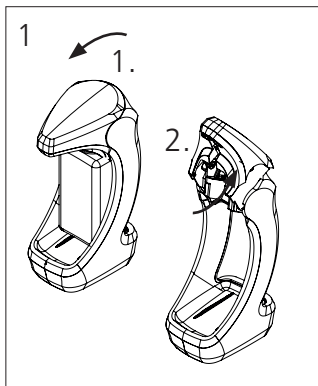
To open battery cover slide the cover towards the back of the dispenser and then rotate cover upwards to expose the battery compartment (fig. 1).

Insert batteries as per marked on the battery compartment (fig 2).

Pay attention to the polarity.

Place the ribbon below the batteries before inserting to aid removal.

NOTICE Batteries should be removed if it is not used for longer periods.



7.2 Inserting the pump & bottle



Open the faceshield by pulling forward (1.) and rotating it upwards and towards the rear of the dispenser (2.) (fig. 1).

NOTICE Remove the filling media before transport!

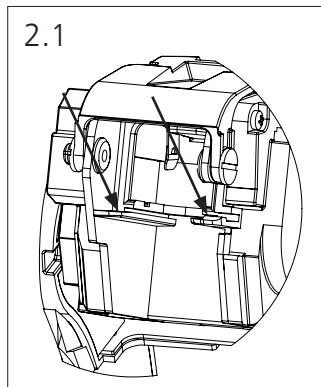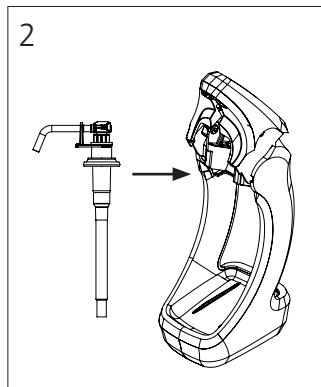EN 7. Going into operation

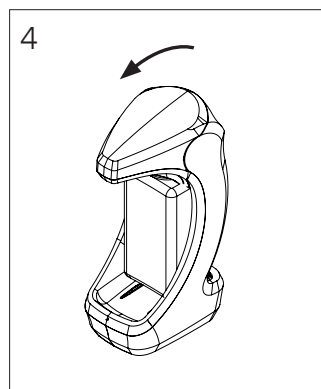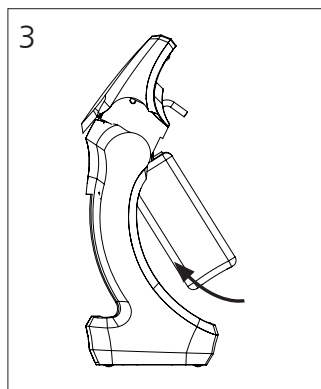Slide the pump into the chassis guides and past the lock tab as shown (fig. 2 and fig. 2.1)




With the bottle an angle slide the bottle up onto the pump
Push down on the front of the faceshield ensuring that it has closed correctly (see that back and sides match)

EN 8. Basic operation

8. Basic operation

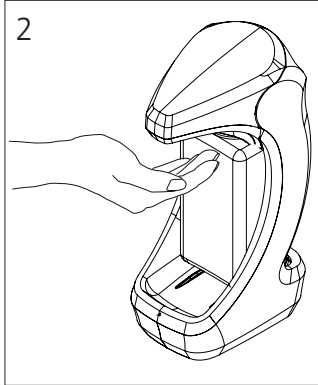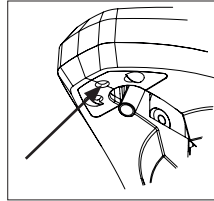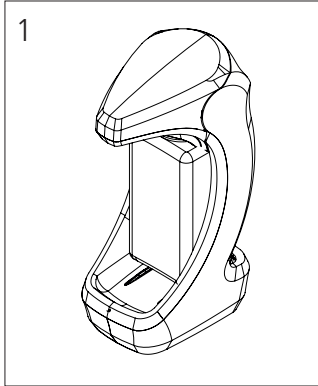Once the faceshield is closed wait 3 seconds for red LED to go off.

To activate the dispenser place the hand underneath the faceshield (approx. 5cm).

This will trigger an automatic dispense. If the unit is set for multiple strokes (2 or 3) it will continue as long as the hand is present .

Dosage: 1,5 ml/ Stroke

EN 9. Changing stroke / 10. LED Function

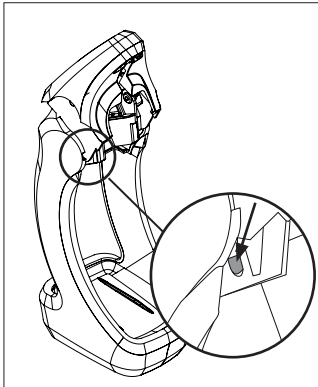
9. Changing stroke

The dispenser can be set to dispense a single, double or triple dosage. If set for multiple dosage the dispenser will only operate as long as a hand is present. If the hand is removed it will complete the current activation and stop the dispenser.

The dispenser is supplied set to single stroke as standard.

With the faceshield open press the tab (left hand side) to increase the stroke


The LED will flash amber when the tab is pressed correctly.

Pressing the tab	Amber flash	Stroke
1st time	2 flashes	2 stroke
2nd time	3 flashes	3 stroke
3rd time	1 flash	Returns to 1 stroke

10. LED Function

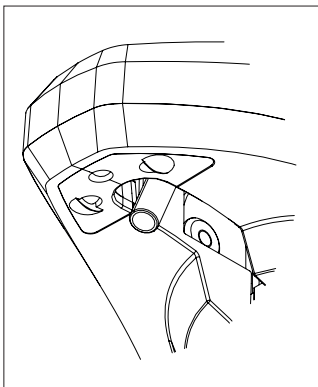Underneath the faceshield there is an LED which lights green, red and amber.

Colour	Explanation
RED (solid)	Faceshield open/not closed correctly
RED (Flashing)	Batteries low
Green	Unit dispensing
Amber	Stroke selection (based on number of flashes)

EN 11. Maintenance / attendance and maintenance

11. Maintenance / attendance and maintenance

Please consider that also a high-quality product like this appliance requires attendance and care. The dispenser itself is maintenance-free with the exception of the battery exchange!

Please follow the recommendations of the Robert Koch-Institute (www.rki.de)

Please only use hot water and do not use caustic purifiers or scouring powders!

Cleaning dispenser:

Remove the mounting plate and dismount the batteries as it is described under 7.

Cleaning of the dispenser only from the outside. Wipe the dispenser with a humid cloth.

 • **By no means disassemble the appliance!**

 • **If the receptacle is filled up, a non-use during a longer period of time does not lead to an impairment. For that reason exchange the empty receptacle in time.**

 • **Please pay attention that the appliance is not submerged or exposed to water jets.**

 • **The appliance is not autoclavable and cannot be cleaned in the dishwasher.**

Cleaning pump:

 • **The pump itself is dishwasher-proof (60°C), however it must rest for about 6 hours after such a treatment before it is inserted again into the appliance.**

 • **If the suction valve is nevertheless clogged, please only pump hot water through it in order to restore the function.**

EN 12. Eliminate trouble

12. Eliminate trouble

 If the appliance malfunctions, please remove batteries or unplug it from the mains.

Characteristic	Possible cause	Solution
<p>1. The appliance seems to function normally but no medium penetrates.</p>	<ul style="list-style-type: none"> No receptacle is inserted. No pump is installed. Appliance is not closed Pump is clogged. Receptacle is empty. Pump is not primes for operation or lost contact to the medium. An unsuitable medium has been used. 	<ul style="list-style-type: none"> Insert full receptacle. Insert pump into the dispenser. Close the appliance Remove pump and clean it. Replace receptacle. Carry out the dosage process several times. If it doesn't function, the pump is possibly clogged. Use another medium. Clean the pump thoroughly before.
<p>2. The appliance doses too much or not enough medium</p>	<ul style="list-style-type: none"> Stroke adjustment is not adjusted correctly. 	<ul style="list-style-type: none"> Adjust the number of strokes. See section 9.
<p>3. The appliance doesn' t dose</p>	<ul style="list-style-type: none"> Batteries are empty Batteries have not been inserted correctly. 	<ul style="list-style-type: none"> Insert new batteries. Pay attention to the polarity

13. Warranty

2 years manufacturer-warranty from the date of delivery.

14. Disposal

Dispose of the packing material into the corresponding recycling plants.

The symbol on the product or its packaging indicates that this product should not be treated as standard household waste, but should be taken to a recycling point for electrical and electronic devices.

Further information may be obtained from your municipality, the municipal disposal companies or the company from where you purchased the product.

14.1 Battery disposal

Batteries should not be disposed of as household waste. As a consumer, you are statutorily obliged to return used batteries. You can dispose of old batteries at domestic collection points, public collection points in your municipality or at any location where batteries of this type are sold.

EN 15. Technical data

15. Technical data

Housing.....ABS

Receptacle.....synthetic material,
removable 500 ml

Pump.....PP, PE & Silcon

Dosage.....1,5 ml/stroke

Voltage.....6 volt DC

Batteries.....4 x LR 14

Dimensions in mm.....B 102 x H 269 x T 142

Total weight.....0,930 kg
(with batteries, without backplate)

Weight.....1,067 kg
optional with backplate

Conformity.....RoHS compliant


CE Machinery directive 2006 / 42 / EG

EMC- directive 2004 / 108 / EG

EN 55014-1:2006

EN 55014-2:2001

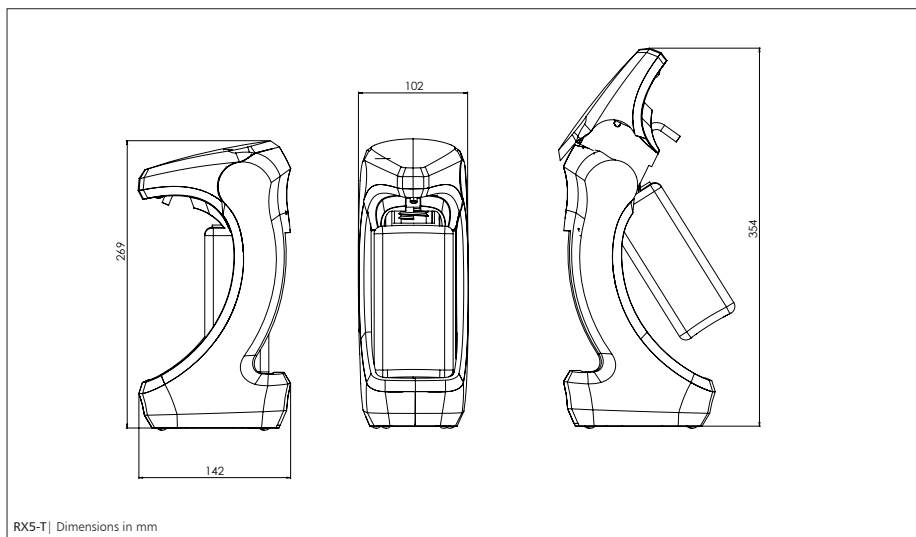EN 60335-1/A11:2014

Technical modifications reserved.
Prior to the use of new filling media tests are required!

OPHARDT
hygiene

OPHARDT Hygiene-Technik GmbH & Co. KG
Lindenau 27 | 47661 Issum | Deutschland
Tel.: +49 2835 18-10 | Fax: +49 2835 18-77
www.ophardt.com | E-Mail: customer care@ophardt.com