## INSTRUCTIONS FOR USE

### SAMPLE CONTAINERS:

TRADE NAME	Cell counter cup, Coulter®. Liquid scintillation vials®. Sample cup for Boeringer® ES 300, ES600. Sample cup for Technicon®, Beckman®, Konelab®. Sample cup Centrifichem®, Technicon®, Beckman®. Cuvettes for Olli®, Cuvettes for Fibrintimer®. Cuvettes for Amelung®. Cuvette segments for Cobas Mira® Microcentrifuge tubes with caps for Cobas Bio® and Cobas Mira®
IDENTIFICATION	<b>SAMPLE CUPS – SEGMENTS – TUBES AND CUJETTES FOR AUTOMATIC ANALYSERS</b>
INTENDED USE	Device for professional laboratory analysis use <b>as container for use in analysers</b> , instruments that automatically perform analysis procedures on an appropriately prepared and packaged biological sample. . Automatic analysers allow an analysis process whose <b>measurements</b> may be made up of one or more numerical values.
NON AUTOMATED DEVICE	-----
QUANTITATIVE DEVICE	The qualitative results of analyses produced by automated analysers must always be subject to the scrutiny of highly qualified personnel for validation and reporting.
SAMPLE	Biological sample or sample of another nature
IN VITRO MEDICAL DIAGNOSTIC DEVICE	Compliant with (EU) regulation 2017/746 for in vitro medical-diagnostic devices
USERS	Highly qualified personnel. For professional laboratory analysis use
STORAGE	Protect from direct sunlight and UV rays. Store in a dry location at room temperature
MATERIAL STABILITY	The plastic materials used allow the product to remain stable over time. It is established that these types of devices are valid for 5 years from the production date listed on the label.
USAGE INSTRUCTIONS AND RESTRICTIONS	comply with laboratory and/or environmental protocols in which the item is used and with the instructions of use for the device used for the analyses use a transfer pipette to transfer an appropriate volume of sample into the container check that the container does not contain any air bubbles make sure to close the container well to prevent contamination and evaporation report any defects that impede usage to the product distributor report serious accidents to the proper authorities dispose of the containers according to the applicable law wear gloves and personal protective equipment when handling potentially contaminating samples keep out of children's reach
SINGLE USE	② do not re-use. Danger of contamination

Kartell SpA  
Via delle Industrie 1  
20082 Noviglio (MI)  
Tel. 02/900121  
Website:[www.kartelllabware.com](http://www.kartelllabware.com)

to access instructions in other languages:[www.kartelllabware.com/en/download/](http://www.kartelllabware.com/en/download/)

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